

PRISM EXOPHTHALMOMETER



The Marco **PRISM EXOPHTHALMOMETER** is designed for accuracy and simplicity in the measurement of the exophthalmos.

Prisms are made of clear plastic with easy-to-read millimeter lines on both sides. The eye is seen through the lower half and the scale through the upper, eliminating the difficulty of parallax and facilitating a rapid determination of protrusion.

The Marco Prism Exophthalmometer has a silver hammer-tone finish and comes complete with a storage case with red velvet liner.

INSTRUCTIONS FOR USING THE MARCO EXOPHTHALMOMETER

- 1) Place notches on the bony lateral orbital margins of the patient.
- 2) Patient is then asked to fixate on a point on the examiner's forehead.
- 3) The apex of the cornea of each eye is superimposed on the millimeter scale on the prisms.
- 4) Align the two red lines on each prism to determine and record the measurement.
- 5) Note: The distance along the horizontal bar should be recorded so that the prisms can be accurately reset for comparison of future readings.



Marco-Certified Quality assures you of the very finest vision products, offering advanced technology, industry-leading durability and enhanced productivity for your practice. Marco-Certified Quality is your mark of confidence.



PLAY IT SAFE.

Protect your warranty. Buy your Marco product only from an authorized Marco distributor. Call Marco for the name of the distributor nearest you.

NO42000 BR-PRISMEXO

T H E L E A D E R I N V I S I O N D I A G N O S T I C S™



11825 Central Parkway | Jacksonville, FL 32224 | 800 874-5274 | 904 642-9330 | www.marcooph.com